

Patient Positioning Techniques

Exam Table Methods

1




Preferred position – side*

2



Alternative position – side*


3

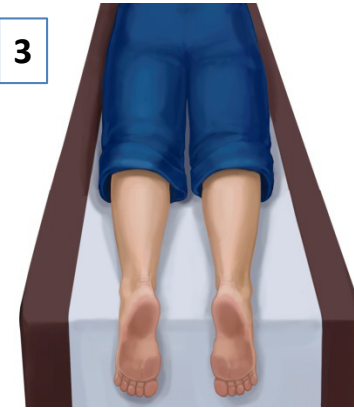

Alternative position - prone

Chair Method

4

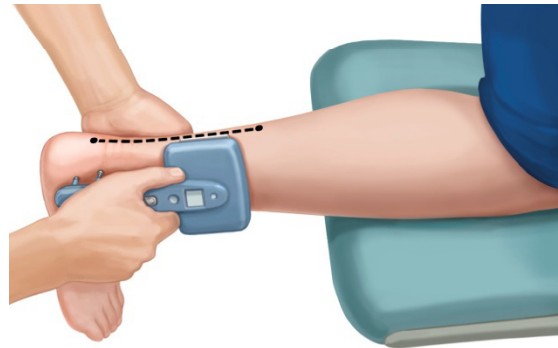

Alternative position - leg up

Check Your Position

Tester:

- Patient's outer ankle bone and Achilles tendon are visible
- Stable grip on device

Patient:

- Fully relaxed upper and lower leg

Conditions above not met? Adjust position or try a different one.

The illustrations above demonstrate the recommended patient positions for testing with the NC-stat DPNCheck device. Patients must be in a comfortable position that allows for relaxation of the leg and foot. Please call Customer Support at (888) 786-7287 with any questions.

*Test leg on top.