

Patient Positioning Techniques

Exam Table Methods



Preferred position – side*




Alternative position – side*



Alternative position - prone

Chair Method

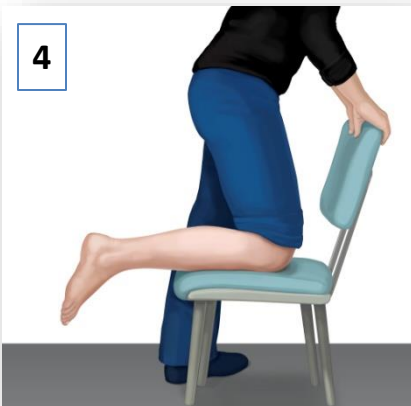

Alternative position - leg up



Check Your Position

Tester:

- Patient's outer ankle bone and Achilles tendon are visible
- Stable grip on device

Patient:

- Fully relaxed upper and lower leg

Conditions above not met? Adjust position or try a different one.

The illustrations above demonstrate the recommended patient positions for testing with the DPNCheck device. Patients must be in a comfortable position that allows for relaxation of the leg and foot. Please call Customer Support at (888) 786-7287 with any questions.

*Test leg on top.